



Printable Measuring Tape

WHAT YOU NEED:

Printer, Scissors, Clear tape

HOW TO ASSEMBLE:

1. Print the measuring tape
2. Before cutting, use a ruler to check the accuracy of the tape. Its important that you follow the printing instructions below.
- 3 Cut along the dotted lines.
4. Place the right hand side of each strip on top the corresponding left handside of the following strip.
5. Use clear tape to attach the pieces in order.

IMPORTANT: MEASURING TAPE MUST PRINT IN ACTUAL SIZE

To be sure it is printed correctly use a ruler to measure one inch to make sure its accurate.

MEASURING TIPS:

For an accurate head measurement, place the measuring tape across the forehead of the baby and measure around the full circumference of the babies head.

SIZE	HEAD CIRCUMFRENCE
Small/0-2 months	13 - 15"
Medium/ 2-4 Months	15 -16.5"
Large/ 4-6 months	16.5 -18"

*Sizing chart fits: Turtle, Adjustable Turtle, Turtle Air

